

Electrotherapy with the NexWave™

The benefits of Electrotherapy as a conservative treatment approach for chronic and acute pain is more important than ever. Consider this option to other non-invasive therapies.

NexWave™

The Wave of the Future in Pain Management

- Decrease Chronic Pain
- Increase Daily Activity
- Improve Muscle Function
- Use at Home, Work, On the Go!
- Safe and Effective Control of Pain

Zynex Patient Support

Zynex will bill your health insurance company for the monthly rental and/or purchase of the NexWave device. Monthly supplies will be shipped to you and billed to your insurance company. Depending on your insurance coverage, you may be responsible for an unmet deductible and/or co-insurance amounts.

If requested, return NexWave directly to Zynex.


Please contact Patient Support for:

- NexWave Technical Support
- Billing Questions
- Monthly Supply Orders
- How to Return Your Device

Contact Us
(800) 495-6670

Made in USA 

Quick Reference Electrode Placement Guide

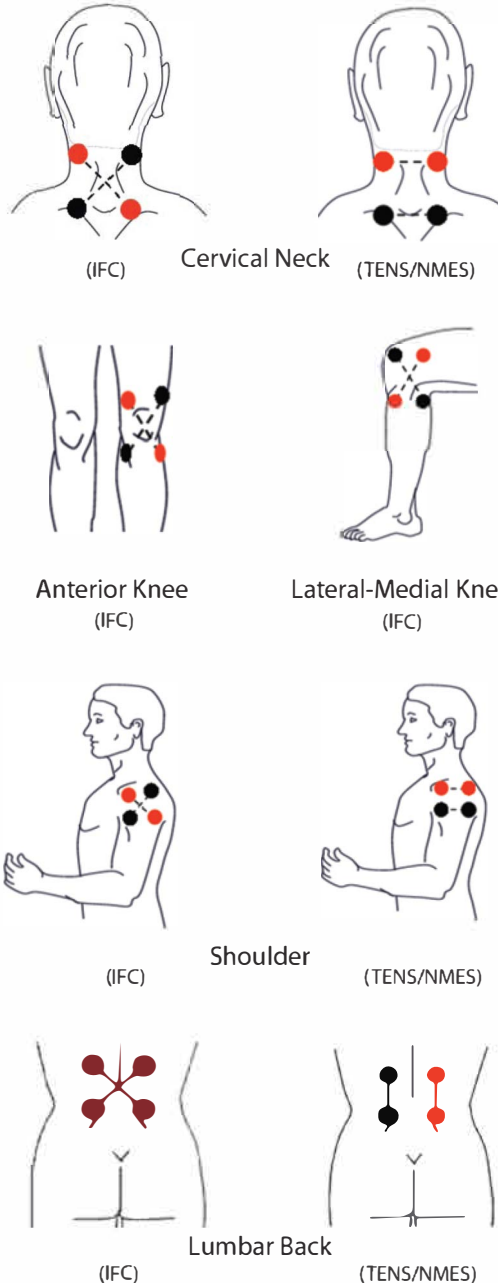

Quick Start Guide

NexWave™



View Instructional Videos:
www.zynex.com/nw



ZYNEX
MEDICAL

Step One

Slide open back compartment & insert 9V battery.
A/C adapter can be used in place of battery.



Step Two

Insert grey & black leadwires into the top of the NexWave™.
Align flat side of leadwire with flat side of device connector.




Step Three

Press middle button to turn device On or Off.




Step Four - IFC

Press the lower left 'IFC' button until the desired mode is indicated by the arrow.

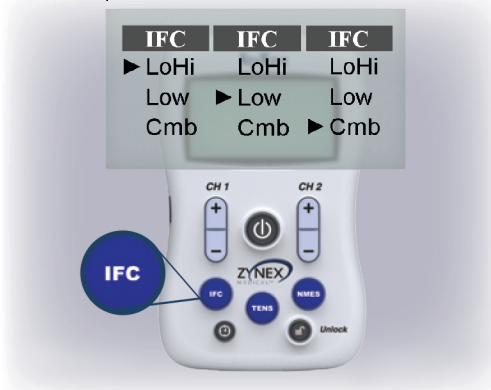


Step Four - TENS

Press the lower middle 'TENS' button until the desired mode is indicated by the arrow.

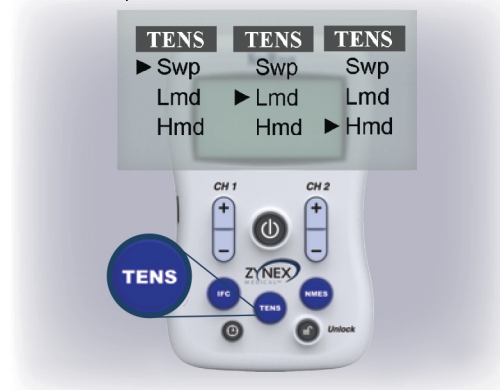


Step Four - NMES

Press the lower right 'NMES' button until the desired mode is indicated by the arrow.

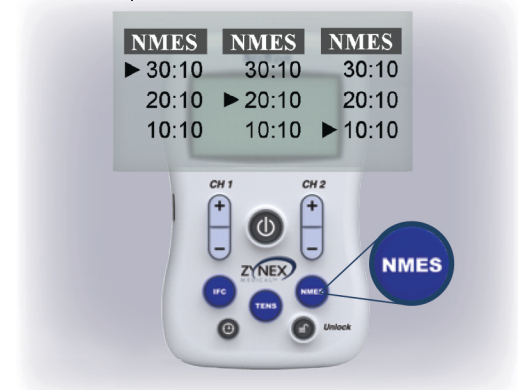

Step Five

Surround pain area with four electrodes, two in each channel. Cross the red and black leads for IFC placement.

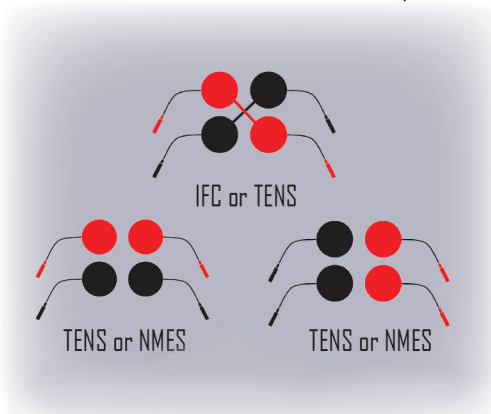

Step Six

Press the lower left timer button until you reach desired treatment time. Press the unlock button to change settings when lock icon is displayed on screen.



Step Seven

Increase intensity by pressing the '+' bar on either aide to a strong, but comfortable, sensation. Press '-' bar to decrease intensity.

